





MRI compatible drip stand

IH MRT und IH HV MRT

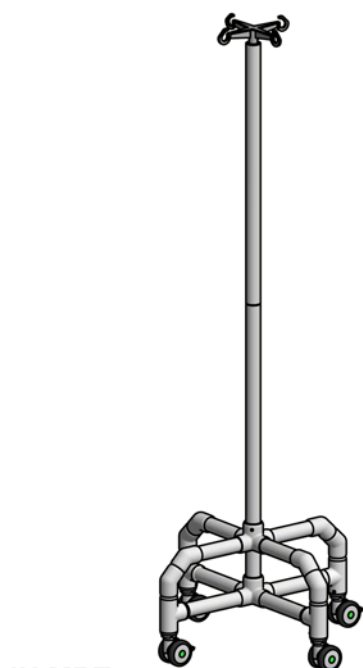

IH MRT

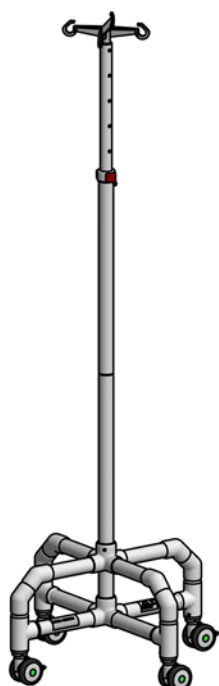

IH HV MRT

Content

1 General and entry control	1
2 General product information	1
3 Special notes	1
4 Operation and functions	2
5 Technical data	2
6 Cleaning and disinfection	2
7 Maintenance and inspection	2
8 Storage, reuse and disposal	3
9 Warranty / Liability	3

1 General and entry control

Congratulations! Today you will receive a quality product from RCN. Please read these operating instructions before using the appliance. It contains important information and safety instructions.

Incoming inspection: Every RCN product is inspected at the factory before delivery to ensure that it is free of defects. However, you should still check the product immediately after receipt - ideally in the presence of the deliverer - for any damage that may have occurred during transport. If you find any such damage, ask the deliverer for an ACCEPTANCE OF DEFECTS (he is obliged to do this) and send this to us together with a DECLARATION OF DISCHARGE (with which you assign claims to us) and the transport documents.

We will then take care of everything else. Damage that you only discover after acceptance is complete is difficult or impossible to enforce against the carriers. Therefore, please take the time necessary for the incoming inspection. As a rule, you will receive RCN products fully assembled and ready for use. If one or more parts are enclosed loose with the product, e.g. to avoid an unnecessarily high transport volume or for adjustment on site, please be sure to observe the markings attached for correct assembly and ensure proper assembly.

Important: If you receive the product in winter and it may have been exposed to sub-zero temperatures during transport, do not load it until it has reached room temperature.

2 General product information

Use the product only on flat surfaces indoors. The model can be permanently exposed to temperatures between 0 °C and 60 °C. Due to the properties of plastic, damage may occur if the product is subjected to loads outside this temperature range. Do not expose the product to selective heat loads such as radiant heaters. Check the product for damage before each use. If there is damage, do not use the product until it has been repaired.

3 Special notes

The product is suitable for MRI areas with devices up to 3 Tesla.



3T
or
Less

4 Operation and functions


The product is equipped with a base with a total of four free-spinning 75 mm plastic brass double-wheel castors. Two castors have a locking brake to prevent the product from rolling away. The bottle cross with four hooks can be loaded with a total of 4 kg. A load of up to 2 kg is possible per hook.

The model with additional height adjustment can be adapted to the individual size of the user. During height adjustment, the bottle cross must not be loaded.

First, the red safety clip must be opened (1) to move the inner tube up and down (2). At the desired height, the safety clip can be closed again so that the tube locks into the next hole.


A: Safety clip closed

B: Safety clip open

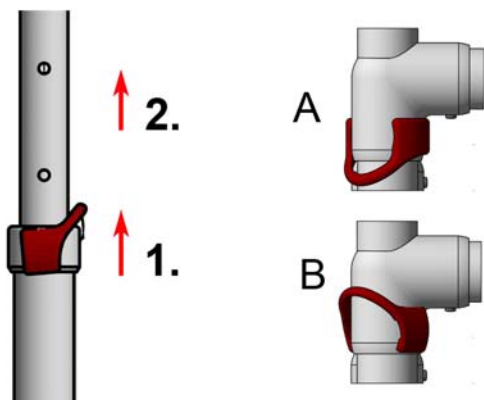

5 Technical data

All dimensions are in millimetres.

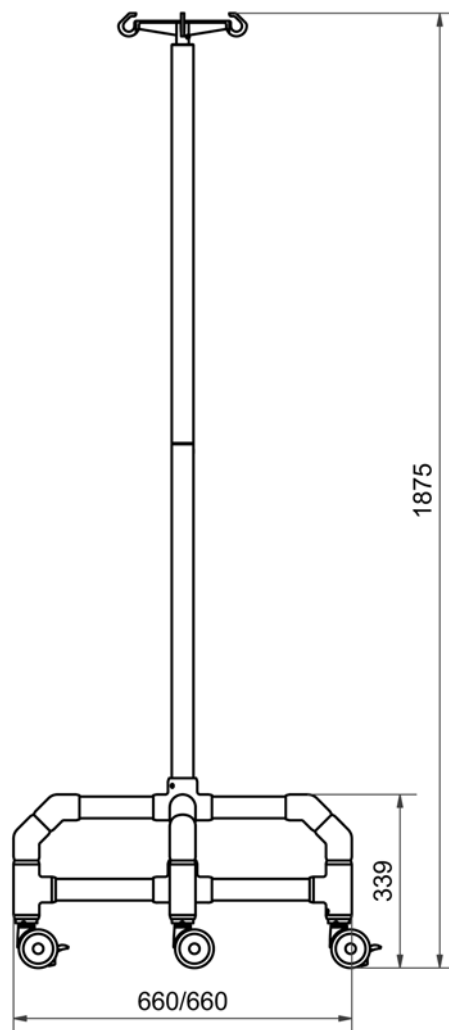


IH MRT

Weight:

5 kg

Max load capacity per hook

2 kg

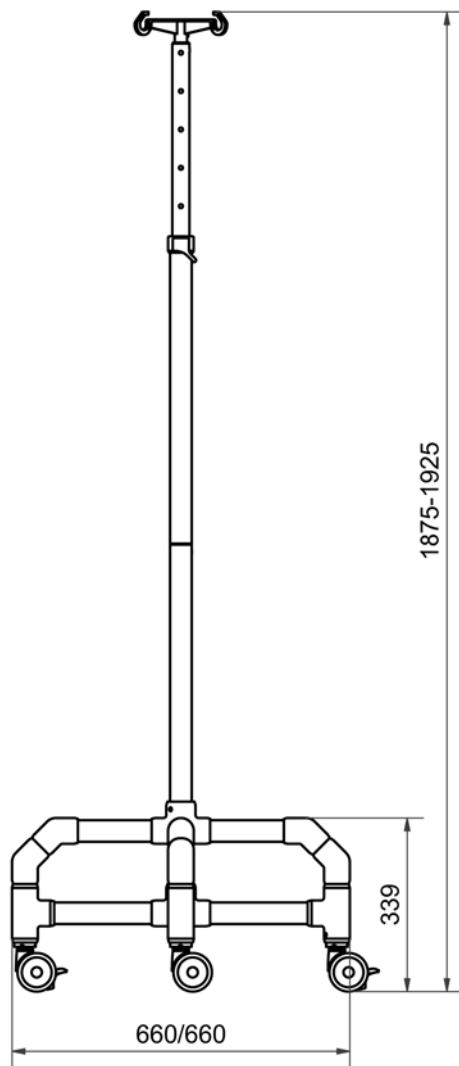

IH HV MRT

Weight:

5,5 kg

Max load capacity per hook

2 kg

6 Cleaning and disinfection

The product can be cleaned easily with warm water and standard household cleaning agents. Use a sponge or soft cloth for this purpose. Caution: Do not use steel wool or scouring agents! For disinfection, use colourless agents suitable for plastics..

7 Maintenance and inspection

The product is maintenance-free if the described cleaning and safety instructions are observed.

A complete inspection must be carried out at least once a year. The correct and tight fit of the connections must be checked. All connecting elements must also be checked for cracks. If necessary, the product must be taken out of operation until after the repair.

The running and turning characteristics of the castors and the function of the parking brakes must be checked weekly.

8 Storage, reuse and disposal

Storage: Store the product at temperatures from 0 °C to 40 °C in a dry environment up to max. 75% humidity. The model cannot be disassembled or folded.

Reuse: The product is suitable for repeated use. The frequency depends on wear and functional capability. Clean and disinfect the product according to these instructions for use. Only pass on the product if all components fulfil their functions.

9 Warranty / Liability

We provide the end user with a guarantee for the RCN product supplied under the following conditions:

The warranty period is generally 3 years, for the plastic or metal frame it is 5 years. It begins at the time the product is handed over to the user. The proof of purchase in connection with the indication of the serial number applied to the product is valid as proof of guarantee.

The warranty covers the repair of damage or defects to the product occurring within the warranty period that are demonstrably due to material or manufacturing defects. This does not include parts that are subject to natural wear and tear. We reserve the right to provide warranty service by replacing or repairing the damaged parts. The guarantee does not cover damage resulting from improper handling and use, failure to observe the instructions contained herein, the use of unsuitable or abrasive cleaning agents or in the event of wilful damage.

In the event of a warranty claim, please contact your specialist dealer/retail partner or RCN directly. If the product is sent to us for repair, this will be at the expense and risk of the sender.

Justified warranty services are carried out without any charge. They neither extend nor restart the warranty period for the product.

Any further claims, in particular those for compensation for damage not caused to the product, are excluded. This warranty is in addition to and in no way affects any statutory or other rights of end users.

We consider it our primary task to develop and manufacture products that best meet the needs of our customers. Tell us if you think we should change or improve something and if you are satisfied, tell us that too, because that is of course what we are most pleased about. You will find our email address, telephone number and fax number as well as our postal address on the first page.

Your RCN Team

Status: 21.11.2022