




Multifunctional shower and toilet chair

SCC 250 PPG SB AP ER

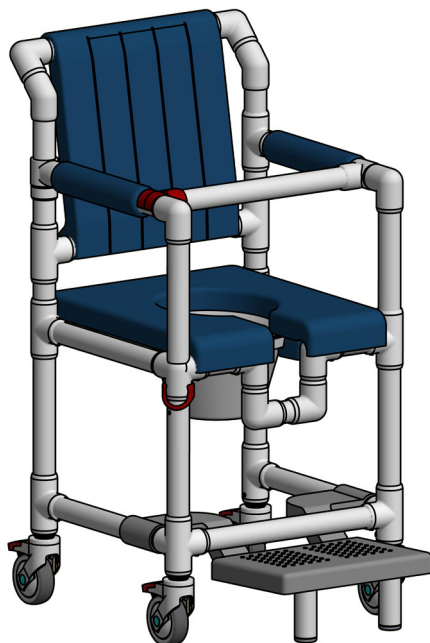

Contents

1 Introduction	1
2 Intended use	1
3 General safety instructions	1
4 Operation and functions	1
5 Accessories	3
6 Technical data	3
7 Labelling	3
8 Cleaning and disinfection	4
9 Maintenance and inspection	4
10 Storage, reuse and disposal	4
11 Warranty / Liability	4

1 Introduction

These instructions for use contain important information on the use of the product. Read the instructions for use carefully and follow the safety instructions in particular.

2 Intended use

The model is intended as a medical device for showering, using the toilet and personal hygiene. The mobile chair with footrest also serves as a transport chair within the bathroom. The backrest and armrests ensure a stable sitting position. Indication: The product provides support for people with limited mobility or balance. It is intended for use by adults and adolescents. Contraindication: Inability to sit.

Product life: The expected service life is 10 years. Individual components (e.g. wheels and cushions) may need to be replaced earlier depending on wear and tear. The service life can be extended by using replacement parts.

3 General safety instructions

Only use the product on level surfaces indoors. Avoid moving around independently by shuffling, as your feet could get caught under the frame. Do not shift your weight by leaning your upper body sideways, forwards or backwards, e.g. when reaching for objects. This can cause the model to tip over.

Remove soap residue from the chair after each use to prevent accidents due to slipping.

When moving the chair over a toilet bowl, take care to avoid the genitals of the person sitting on the chair.

The model can withstand temperatures between 0 °C and 60 °C on a permanent basis. Due to the properties of plastic, damage may occur if the model is exposed to loads outside this range. Do not expose the product to concentrated heat sources such as radiant heaters. Check the product for damage before each use. If damage is found, do not use the product until it has been repaired.

All serious incidents involving this product must be reported to us (the manufacturer) and the competent authority.


4 Operation and functions

The product is delivered fully assembled.

• Wheels: 100 mm single-wheel castors

The model is equipped with 4 free-spinning Ø 100 mm single-wheel castors.


All castors have a locking brake. When transferring a person and during use, all locking brakes must be engaged.

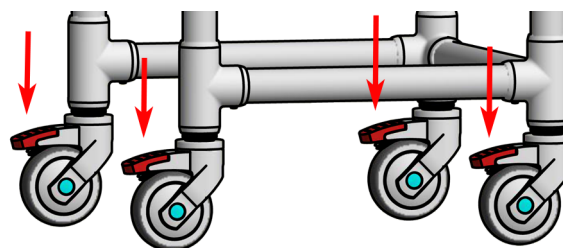

• Slide-out footrest FT

The footrest is used to rest the feet on. The footrest is attached to the lower frame rails in such a way that it can be easily moved into position by pulling it forward or pushing it back.

The support bars with non-slip feet attached under the footrest make the footrest stable and prevent the chair from tipping forward. If larger castors are retrofitted, the support bars must also be extended.

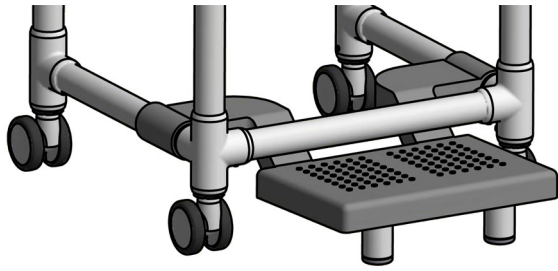


- Swivel armrest ASA

This makes it easier to transfer the user into and out of the chair. To do this, first release the lock (1) by turning the red plastic bracket 90° upwards. Then lift the armrest at the rear of the swivel joint (2) and swing it outwards (3). Locking is done in reverse order. Always lock the armrest after the transfer! Caution: The armrest is not to be exposed to any loads when it is swung out!

A: Safety latch closed

B: Safety latch open

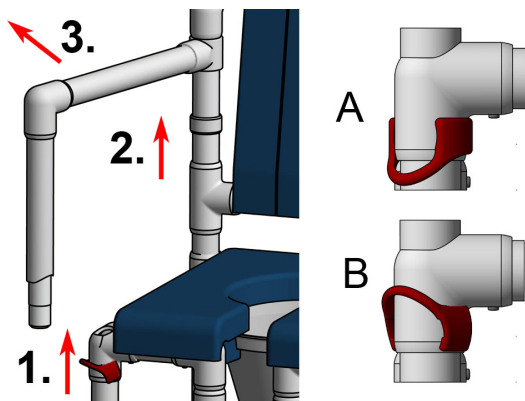

- Safety bar SB

This serves as an additional handhold. The safety bar is attached to an armrest so that it can be rotated. The bar is closed by pressing the half-open end onto the opposite armrest. It is then positioned horizontally in front of the person sitting in the chair. However, it does not constitute a restraint because it can be opened at any time.

The easiest way to open it is to reach under the bar with your hand and press down on the armrest with your thumb.

Please note: The safety bar serves as additional protection during transport and allows the user to hold on to it comfortably. However, the bar does not provide absolute protection against slipping or falling out.



- Padded armrests AP

The polyurethane foam padded armrests increase seating comfort.

- Soft seat PP


The polyurethane foam seat increases seating comfort and also facilitates access to intimate areas.

The seat is simply placed on the seat frame of the chair and can be removed for cleaning. The pail is inserted from the left side on standard seats. The handle must be parallel to the insertion direction!

- Pail TK


The plastic (polypropylene) pail is temperature-resistant up to 105 °C. The pail fits in all potty washers and is resistant to disinfectants. Pressing down on the lid creates a vacuum, which reduces odours. The pail is held in place by placing the fingers of one hand under the rim. It has a scale on the inside to read the fill level. The pail is inserted as shown in the illustration.

Caution: As a general rule, the pail should not be inserted or removed while in the sitting position in order to protect the genitals.

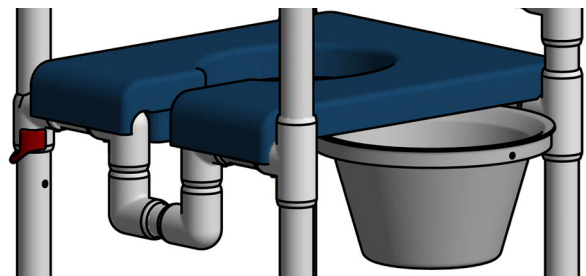

- Soft backrest PR


The soft backrest increases seating comfort. It is simply pressed onto the lower rail and then pressed over the upper rail with the lip pulled up slightly. The soft backrest can be removed just as easily for cleaning. The lid of the pail can be quickly attached to the back by pressing it down.

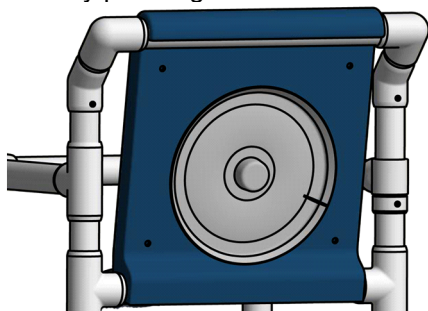




5 Accessories

- PPE insert

The insert is used to cover the seat opening in order to reduce the spread of odours when using the chair as a commode chair and to visually cover the seat. The insert does not create a fully load-bearing closed seat. The insert must be removed when showering or using the toilet.

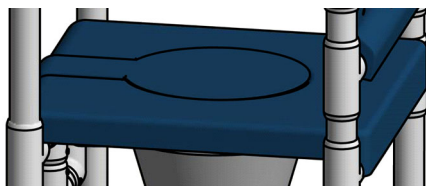

6 Technical data

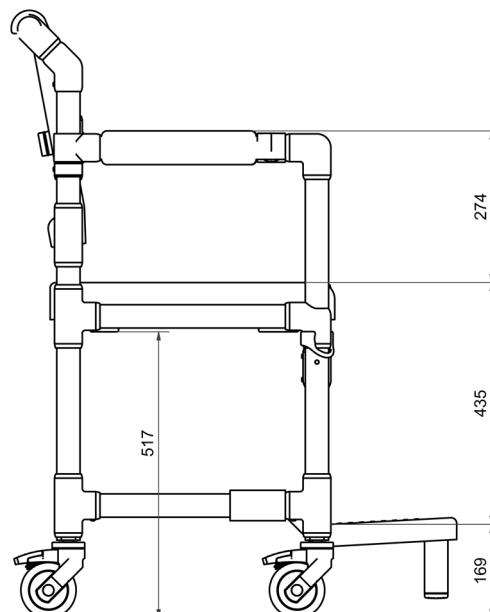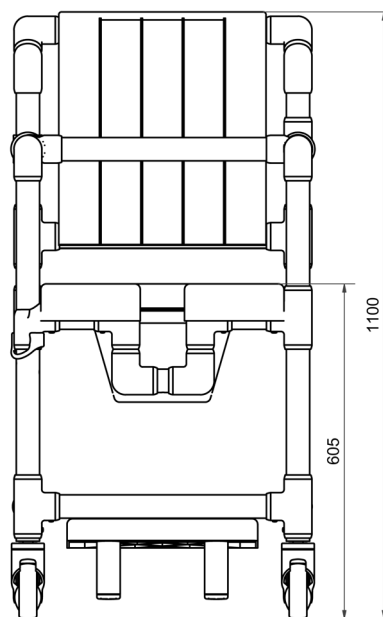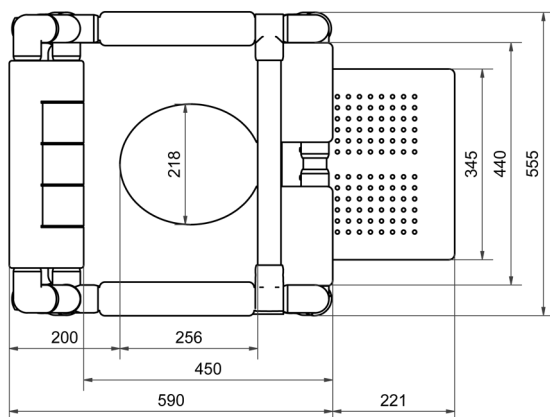

SCC 250 PPG SB AP ER

Weight: 21.5 kg

Maximum load capacity: 150 kg

7- Marking

Labels with the following symbols are affixed to the frame of the product:



Medical device



CE mark



Order number



Serial number



Maximum user weight kg

	Manufacturer
	Date of manufacture YYYY-MM
from 26/05/2027	Example
	Unique Device Identifier Barcode GS1 DataMatrix
	
(01)04260629321092	(21)202105673
(01)GTIN	(21)SN

8 Cleaning and disinfection

The product can be easily cleaned with warm water (max. 60 °C) and standard household cleaning agents. Use a sponge or soft cloth for this purpose. Caution: Do not use steel wool or abrasive cleaners! For manual cleaning, wipe all components with a colourless surface disinfectant that is suitable for plastics. Follow the instructions of the disinfectant manufacturer regarding exposure times and safe use.

The following applies to polyurethane foam cushions: Do not clean or treat with sharp objects. Only use soap-based (alkaline) cleaning agents at temperatures up to 80 °C. Acidic cleaning agents are not suitable.

Caution: When using rapid disinfectants, do not rub the cushions during the application time and for 20 minutes afterwards. The cushions must be cleaned of dirt beforehand if necessary.

To ensure that the slide-out footrest continues to operate smoothly, please clean the gaps as required by sliding the sliding parts back and forth under a strong jet of water. Spraying the sliding surfaces with silicone spray promotes smooth operation.

9 Maintenance and inspection

The product is maintenance-free. Observe the cleaning and safety instructions described.

A complete inspection must be carried out at least once a year. During this inspection, the correct and secure fit of the connections must be checked. All connecting elements must also be checked for cracks. If necessary, the product must be taken out of service until after repair. Technically trained persons can also carry out repairs themselves. Ask for our repair instructions.

Before each use of the product, carry out a visual inspection and a functional test. In particular, check the running and rotating properties of the rollers and the function of the parking brakes.

10 Storage, reuse and disposal

: Store the product at temperatures between 0 °C and 40 °C in a dry environment with a maximum humidity of 75%. The model cannot be dismantled or folded.

Reuse: The product is suitable for repeated use. The frequency depends on wear and tear and functional suitability. Clean and disinfect the product in accordance with Chapter 8. Only pass on the product if all components are in working order.

Disposal: The product does not contain any hazardous substances. You can dispose of the product at your local waste disposal facility.

11 Warranty / Liability

We provide the end user with a warranty for the delivered RCN product under the following conditions:

The warranty period is generally 3 years, and 5 years for the plastic or metal frame. It begins at the time the product is handed over to the user. The proof of warranty is the proof of purchase in conjunction with the serial number affixed to the product.

The warranty covers the repair of damage or defects to the product occurring within the warranty period that are demonstrably due to material or manufacturing faults. This does not include parts that are subject to natural wear and tear. We reserve the right to provide warranty service by replacing or repairing the damaged parts. The warranty does not cover damage resulting from improper handling and use, failure to observe the instructions contained herein, use of unsuitable or abrasive cleaning agents, or wilful damage. In the event of a warranty claim, please contact your specialist dealer/trade partner or RCN directly. If the product is sent to us for repair, this is at the expense and risk of the sender.

Eligible warranty services will be performed free of charge. They do not extend or restart the warranty period for the product.

Further claims, in particular those for compensation for damage not caused to the product, are excluded. This warranty is additional and does not affect the statutory or other rights of end users in any way.

We consider it our primary task to develop and manufacture products that best meet the needs of our customers. Please feel free to share your wishes and criticism with us. You will find our contact details on the first page.

Your RCN team

As of: 23 February 2026